
# HiPPo H1 Quick Guide

<https://www.hippodiabetes.com/>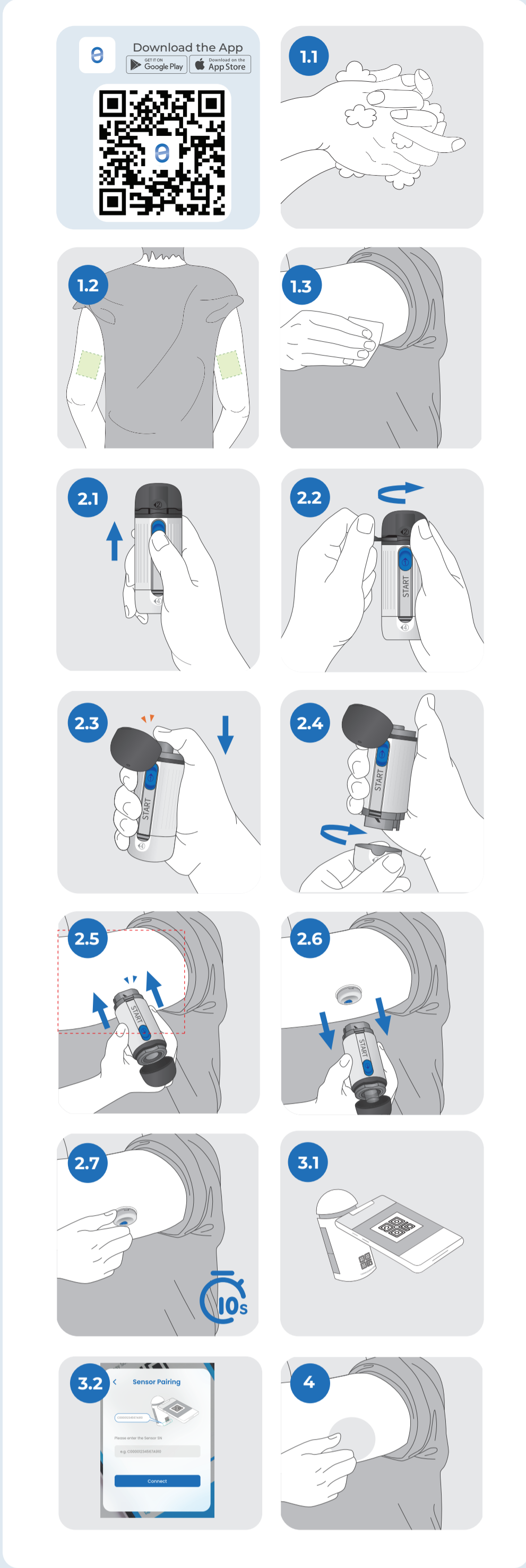

# Start your Sensor

Instruction for Use **IE** English

## Install APP & Login

Scan the QR code to download the APP, and register a HiPPo account in the APP.

### 1. Clean the wearing site

- 1.1 Thoroughly wash your hands with soap and allow them to air dry.
- 1.2 We recommend wearing it on the outer back of the upper arm, as this area typically has more fatty tissue.

**\*The sensor isn't tested or approved for other sites.**

- 1.3 Disinfect the area with alcohol, wipe thoroughly, and allow it to air dry completely.

### 2. Wear the sensor

- 2.1 Slide the blue button to the top position.

**\*Slide the button upward until it reaches its limit, and cannot to be pushed any further, and you will see a silver sealing pop out.**

- 2.2 Remove the sealing ring, and then open the top cap. (You will see a button underneath the cap)

- 2.3 Press the button

**\*Press down until you hear a "click" sound.**

- 2.4 Unscrew the bottom cap

**\*Please do not press down on the applicator until it is placed on the intended site.**

- 2.5 Position the bottom of the applicator against the intended site. Apply firm pressure with the top cap closed, and continue pressing the applicator until you hear a distinct 'click' sound.

- 2.6 Gently remove the applicator vertically.

**Please do not discard the applicator now!**

- 2.7 Ensure the sensor is securely attached to your skin by gently rubbing the adhesive tape around it for 10 seconds.

### 3. Connect your sensor in the APP

- 3.1 By scanning the QR code on the back of the applicator to pair your sensor in the APP.

- 3.2 If you cannot scan the code, you can also input the connecting code manually.

### 4. Use the overpatch

You must apply the overpatch to keep the sensor on your body. It's inside the envelope within the sensor box.

**\*\*review the product instructions that come with your continuous glucose monitoring device. These instructions have important information, including indications, contraindications, warnings, and precautions. Note: You can go to [https://www.hippodiabetes.com/downloadsandguides] or scan the QR code on the bottom of the photo instructions. Or contact your local sales representative to get product instructions.**

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HiPPO BIOSENSORS is a registered trademark of HIPPO MEDICAL PTE. LTD. in Singapore and/or other countries.
All other product or company names that may be mentioned in this publication are tradenames, trademarks or registered trademarks of their respective owners.
**HIPPO MEDICAL PTE. LTD.**
**PureCE B.V.**